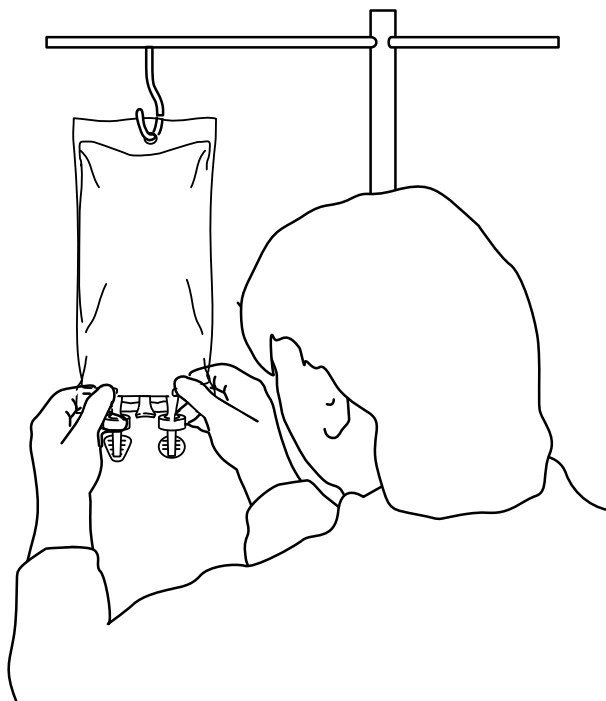
# Ecobag®

- CZ** Návod k použití
- DE** Handhabungshinweise
- DK** Brugsvejledning
- ES** Instrucciones de uso
- FI** Käyttöohje
- FR** Mode d'emploi
- GB** Handling instructions
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- NO** Brukerveiledning
- PT** Instruções de uso
- SE** Bruksanvisning
- SK** Návod na použitie

**B | BRAUN**

black

Dimension: 148 x 210 mm  
18 pages

Pharmacode: - - -

WNr\_\_\_W230934...  
0/UF00100/0523  
GIF  
Production site: Crissier

Font size: 8,0 pt.

W230934

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# GB Handling instructions

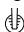
## 1. Preparation of the container

- ① Check container and closure are intact.

Check contents for clarity and colour aberrations.

Open container by twisting off the corresponding toggle.

(  ⇒ Infusion port)

(  ⇒ Additive port)

The opened infusion port site is sterile.

Note:

Connect the transfer device immediately following opening. The port should be disinfected if the closure toggle has already been removed before the time of use.

## 2. Gravity infusion

- ② Use non-vented infusion sets (or keep air vent closed).

Close roller clamp.

Insert spike of the infusion set as far as it will go.

- ③ Fill drip chamber approx. halfway with fluid.

Fill tubing of the infusion set, avoiding air bubbles.

- ④ Remove protective cover from the end of the tubing.

Connect infusion set with the patient.

Start the infusion with air vent closed.

## 3. Pressure infusion

- ⑤ Insert infusion set (Note: use only pressure resistant infusion sets).

Turn port system upwards with infusion set inserted.

Expel air out of the container (roller clamp open) and fill drip chamber approx. halfway with fluid.

Close roller clamp.

- ⑥ Insert container into the pressure cuff.

Apply pressure to the container.

Open roller clamp and start pressure infusion.

## 4. Admixture of additives with syringe

- ⑦ Open the additive port () by twisting off the corresponding toggle. The opened injection port site is sterile.

Add drug mixture (Note: pay attention to possible incompatibilities).

Shake the container to mix the contents thoroughly.

## 5. Admixture of additives with transfer set

- ⑧ Open medication vial.

Disinfect injection site of the medication vial.

For transfer of dry substances attach the transfer set to medication vial first and insert until it fits firmly.

- ⑨ Open the additive port () by twisting off the corresponding toggle. The opened injection site is sterile.

Attach transfer set together with the vial to the additive port and insert until it fits firmly.

- ⑩ Transfer fluid into the medication vial by repeated pressing of the bag (Note: pay

attention to possible incompatibilities).

Shake the medication vial gently until the contents of the medication vial are dissolved completely or mixed thoroughly.

Take care during shaking that bag, transfer set and medication vial are securely connected.

- ⑪ Hold bag, transfer set and medication vial together and turn upside down.

Transfer the drug solution back into the bag by pressing the bag repeatedly. The procedure may be repeated to ensure a complete transfer of the drug solution.

Remove transfer set together with medication vial and dispose of them.

Shake container to mix contents thoroughly (Note: pay attention to complete dissolving of dry substances as well as to possible incompatibilities).

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**IT** **B. Braun Carex S.p.A.**
**IE** **B. Braun Medical Ltd.**
**SE** **B. Braun Medical AB**
**NO** **B. Braun Medical A/S**
**NL** **B. Braun NPBI Medical B.V.**